




# OrCam MyEye 3 Pro Smart Magnifier Quick Start Guide

## Set up

1. Power on the *OrCam MyEye 3 Pro* device.
2. Connect the *OrCam MyEye 3 Pro* to your computer using the supplied micro-USB cable with its magnetic connector.
3. Place the *OrCam MyEye 3 Pro* on the supplied stand.
4. On your computer, browse to [cast.orcaml.com](http://cast.orcaml.com)
5. Press "Start".
6. Create an account using your Google or Apple Account or your name and email address.
7. Click "Connect via USB" to connect your computer to the *OrCam MyEye 3 Pro* device.

## Using the Smart Magnifier

1. Place a book or any piece of text under the *OrCam MyEye 3 Pro* camera.
2. Align the piece of text so that all of it is visible.
3. Press the star button in the center of the button bar.
4. If requested, please allow access to your Camera and Microphone. These permissions are required for the web application to work properly.
5. The *OrCam MyEye 3 Pro* will snap a picture of the text and process it. Then, it will play a summary of the text.
6. You can ask any questions regarding the text or any general questions.
7. If you want the magnifier to read the text aloud, press the "Audio Book" icon to switch to text reading mode.
8. Use the left side toolbar to zoom in and out, switch between text and image modes, rotate the image, change contrast, and adjust the brightness.
9. Press the download button to save the captured image, copy the text, and download the audio transcript file.

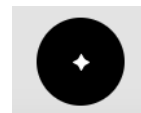

## Upload file

1. You can upload different types of files that contain text and use the magnifier to read them out loud.
2. Press the + icon and select "upload file".
3. Select the desired file from your computer.
4. The file's content will appear on the screen, and you can use the magnifier to interact with it.