



OrCam MyEye Pro

Quick Instruction Guide

Basic Instructions

To Power On/Off: To turn the device on, press and hold power button for 2 seconds. To turn the device off, press the power button until you hear “Suspended, press again to shut down.” The power button is located on the inside of the long side of the device next to two magnets. You can move your finger along this side of the device to feel them.

Volume Up/Volume Down: To increase volume, swipe forward on the touch bar when the device is not reading. To decrease volume, swipe backward on the touch bar when the device is not reading.

Access the Set Up Menu: Double tap the touch bar when device is not reading. Or after the beep, say “Hey OrCam, Enter User Menu” to access menu settings. Swipe forward/backward to navigate through the menu options.



Reading

OrCam MyEye can read printed and digital text from any surface: i.e. books, bank statements, newspapers, smartphones, computer screens.

Start/Stop Reading: Initiate reading by centering the text in front of your nose and then either tapping the touch bar or pointing your finger at the text to read. When you hear a double beep, remove your finger. To stop reading, tap the touch bar or use the “stop” hand gesture.

Reading Navigation: Double tap the touch bar to pause or resume reading. To fast forward while reading, swipe forward on the touch bar. To go backward while reading, swipe backward on the touch bar.

Facial Recognition

OrCam MyEye is able to identify friends, family and loved ones in real time. You can store up to 100 recognizable faces

- To save a face using voice commands, say “Hey OrCam, Start Face Learning.” Say the name of the person and then say confirm.
- To save a face manually: Hold down on the touch bar while aiming toward a person standing about 3 feet away. After 15-30 seconds release your finger from the bar. State the person’s name after the beep and then wait. It will beep again and repeat the recorded name back to you. Tap once on the touch bar to finish saving this person, swipe to record name again or double tap to cancel.
- The next time your OrCam MyEye sees this person, it should acknowledge them by the recorded name you put in.

How To Recognize Barcodes and Money

OrCam MyEye is able to recognize preloaded barcodes and money notes.

- To recognize a product or identify money note, aim the device at the barcode or dollar bill, holding it about 12 inches from the glasses. Make sure the object is aligned centered with your nose.

Color Recognition

Use OrCam MyEye to identify colors to help with selecting clothing or other items.

- Point finger at object, after a few seconds color will be identified.



Smart Reading

Use the Smart Reading feature to easily find specific information you are looking for. You can ask OrCam MyEye to read the due date and amount of your bills, read just the desserts section of a menu, read the phone number off a business card, and more.

- Hold the document in front of you while the camera is directly facing the text.
- Use the voice command “Hey OrCam, Start Smart Reading” or double tap the touch bar and say “Smart Reading.”
- Hold still until you hear the camera shutter sound. The device will say “Ready” and you can begin Smart Reading using the available smart reading vocal commands – listed below.
- To say a new command while the device is reading, tap the touch bar once and the device will return to: “Always listening mode”.
- To exit this feature: Say “Exit” or hold the touch bar for 5 seconds and release.



Smart Reading Commands

Start by saying “Hey OrCam” followed by one of the commands below:

- Start Smart Reading
 - Find “X” (for example: “Find Dessert”)
 - Start from “X”
 - Jump to “X”
 - Read Everything
 - Restart (will capture a new picture)
 - Previous
 - Next
 - Repeat the last sentence
 - Read the phone numbers
 - Read the dates
 - Read the amounts
 - Read the headlines
 - Read the article number “X” (can be a number 1-10)
 - Read the article about “X”
 - Speak louder
 - Speak softer
 - Speak faster
 - Speak slower
- Exit


Voice Commands

Start by saying “Hey OrCam” followed by one of the commands below:

General Commands

- Speed Up/Speed Down
- Volume up/Volume Down
- Tell Time
- Tell Date
- Tell Battery Status
- Connect to Bluetooth Device
- Main Menu
- Read Everything
- Go To Sleep
- Change Voice
- Start Barcode Learning



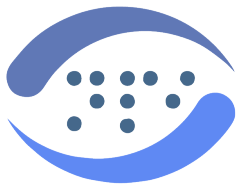

NANOPAC.COM
TECHNOLOGY FOR INDEPENDENCE SINCE 1987

NanoPac, Inc.

918-665-0329

info@nanopac.com

OrCam Devices for Low Vision



OrCam Read



OrCam MyEye



OrCam Read 3



Email: usorcam@orcam.com